noah4

Getting Started

Table of Contents

| | |
|--------------------------|----|
| GETTING STARTED | 3 |
| INITIATION | 5 |
| CÓMO EMPEZAR..... | 7 |
| INICIANDO | 9 |
| BEGINN DER ARBEIT | 11 |
| AAN DE SLAG | 13 |
| PREPARAZIONE..... | 15 |
| ROZPOCZYNANIE PRACY..... | 17 |
| INTRODUKTION | 19 |
| KOMMA I GÅNG | 21 |
| KOMME I GANG..... | 23 |
| ALOITUS | 25 |
| PRADŽIA | 27 |
| ELSŐ LÉPÉSEK | 29 |
| BAŞLARKEN..... | 31 |
| ZAČÍNÁME | 33 |
| НАЧАЛО РАБОТЫ | 35 |
| スタートガイド..... | 37 |
| 入门指南 | 39 |
| 入門指南 | 41 |
| 시작 가이드..... | 43 |

Trademarks Noah is a registered trademark of HIMSA II K/S.

Copyright This document is copyrighted by HIMSA II K/S with all rights reserved. Under copyright laws, this document cannot be reproduced in any form without written permission of HIMSA II K/S.
© 2010 by HIMSA II K/S. All rights reserved.

Getting Started

IMPORTANT: If you do not register your Noah 4 license, the Noah software will stop functioning after 45 days. For information on obtaining a license, please contact your Noah distributor.

Preparing for Noah 4 Installation

Before you install Noah 4, you should:

- Make sure your PC(s) meets the minimum Noah 4 requirements
- Determine which modules your Noah 4 installation will need.
- Ensure you have Administrator rights on the PC

Minimum System Requirements

In order to receive satisfactory performance from your Noah 4 application, we recommend that your PC and network server (if applicable) meet or exceed the following minimum specifications (Please note that Noah-compatible modules often have greater minimum requirements than Noah):

| PC Specification | Minimum Requirements |
|-------------------------------|--|
| Processor/clock speed | Pentium III, 1 GHz or similar |
| System RAM | 512 MB (1024 MB recommended) |
| Minimum free hard drive space | <ul style="list-style-type: none"> • 1 GB of free space for Noah (5 GB recommended) • 1 GB of free space for <u>each</u> installed module • 4 GB of free space for the patient database |
| Operating System | Windows XP Professional SP3 (32-bit) ¹ , Windows Vista SP2 (32- and 64-bit), Windows 7 ¹ <i>Noah Server installations:</i> Windows Server 2008, Windows Server 2003 SP2 |
| Screen Resolution | 1024 x 768 or higher |
| Bluetooth | Required for NOAHlink |
| Adobe Acrobat Reader | This is needed to read the Noah manuals. Go to the Adobe site to download the free Reader |

¹ HIMSA does not support Windows XP Home or Windows 7 Starter.

Determining the Modules You Will Need

Before installing Noah 4, you should determine which measurement and fitting modules you will need, and have their installation files available. Visit HIMSA's certification site (www.himsa.com) for a list of Noah 4-certified modules.

New Noah 4 user: Contact the hearing instrument and audiological equipment manufacturers with whom you regularly work. They can provide you with Noah modules specifically designed for their products.

Upgrading from NOAH 3 to Noah 4 on a PC with NOAH 3: The NOAH 3 modules will automatically be migrated to Noah 4. Visit HIMSA's certification site (www.himsa.com) for a list of Noah 4-certified modules. After the Noah System 4 upgrade, NOAH System 3 will no longer be available on your PC.

Installing Noah & Related Software

1. Place the Noah 4 installation disc into your drive. The installation program should automatically start within 30 seconds. If not, use Windows Explorer to browse the installation disc, and click on 'setup.exe'.
2. Follow the on-screen instructions. After installing, Noah 4 will open automatically. If, during installation, you migrated a NOAH System 3 that was password-enabled, then you will be required to enter your NOAH 3 user name and password. Otherwise, HIMSA recommends enabling password security after logging on – for more details, click on Help in Noah 4.
3. After installing Noah 4, you should install or update your license file – visit HIMSA's support site (www.himsa.com) for further details. If you do not have a Noah license, please contact your Noah distributor.
4. Visit HIMSA's support site (www.himsa.com) to browse for updates.
5. After installing Noah 4, you can begin installing fitting, measurement and product order modules, if necessary.

IMPORTANT: If you have performed a network installation, you will need to install modules on each Noah Client machine.

Information & Support

If you have support questions, please check your Help System. If your problem persists, visit HIMSA's support site (English/German): www.himsa.com. If you still cannot find the answers you need, contact your Noah distributor.

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